



ADTP product information & directions for use



STEP 1:

Open up ADTP belt mounting velcro this will aid when installing your Tourniquet for the first time (ADTP is made on the tighter side and will break in normally after 2-3 reps) At this time you will need to decide whether you want to install the pouch with the access flap in a pull tab up to access or pull tab down to access as well as tourniquet facing forward or rearward. ADTP will accommodate all of these .



STEP 2:

Your tourniquet needs to be folded a certain way to make use of the assisted deploy feature (if you're not using this feature the pouch still functions as tourniquet holder). Pull the strap thru the buckle until the product information printed on the TQ strap becomes visible and press down firmly. Folding it this way allows for the widest opening to clear clothing and obstructions and quick to release the TQ strap back thru the buckle if desired.



STEP 3:

Take the remaining strap and fold over until the top of the fold lies just below the buckle. This completed TQ fold is what will be inserted into the assisted deploy portion of the pouch.



STEP 4:

Insert the completed fold under the provided clear plastic sheet in the orientation you've decided to mount the pouch and push the TQ fold all the way thru until the buckle sits just before the pouch. This will take some effort but will become easier as the pouch breaks in.

IMPORTANT: REMEMBER TO REMOVE THE CLEAR PLASTIC SHEET AT THIS TIME, THIS IS ONLY USED TO AID IN THE SET UP PROCESS



STEP 5:

Complete by folding the top portion of the TQ over and close the access flap, make sure to press down firmly ensuring all hook and loop are locked down.

You're all done with the set up and can now mount the pouch to your belt!

